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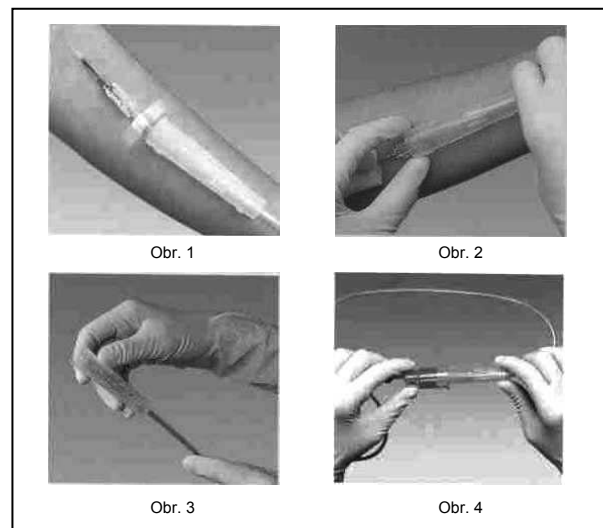
Instructions for Use

WARNINGS

1. The product is sterile and non-pyrogenic only if the unit package is undamaged and all caps are in place.
2. Do not use the product unless solutions are clear and unit package is undamaged.

PRECAUTIONS

1. JMS BLOOD BAG is intended for single use only. Discard entire set after one use.
2. Dispose by acceptable method for potentially contaminated blood products.
3. Be careful to avoid accidental needle sticks when handling donor needle.
4. Do not use BLOOD BAG more than 10 days after breakage of aluminium foil package.
5. JMS Blood Ports are designed to achieve optimal fitting with ISO spike.
6. Position filled bags in centrifuge cups in a manner that bags are firmly sit in cup with minimum dead space.

BLOOD COLLECTION

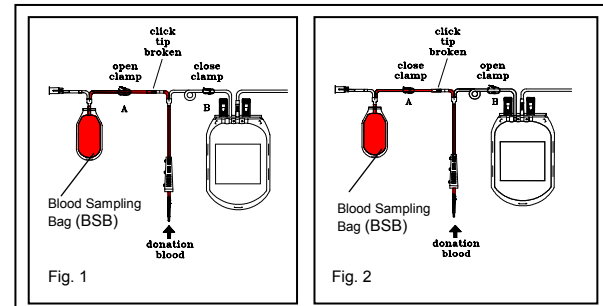
1. Use aseptic technique.
2. Confirm that the size and capacity of BLOOD BAG is suitable for blood collection intended. Blood collection volume is indicated on the primary bag label.
3. Confirm that all numbered tubing of each blood bag unit contains its own identical segment numbers.
4. Adjust blood collection scale/mixer to desired collection.
5. Place or suspend BLOOD BAG on blood collection scale/mixer.
6. Clamp donor tubing just above donor needle.
7. Apply tourniquet on donor's brachium to assist in locating vein. Disinfect site of phlebotomy.
8. Remove needle cover as per instructions below. The black line on the needle hub indicates bevel side:
(a) Grasp needle hub and cover firmly near the connection and twist cover.
(b) Pull needle cover straight off, being careful not to drag the cover across the needle point.
9. Insert needle into vein with bevel up. Release clamp on donor tubing.
10. During blood collection, occasionally and gently invert or shake BLOOD BAG to ensure thorough mixing of blood and anticoagulant.
11. Collect required amount of blood.
12. Clamp donor tubing and stop blood flow when the collection is completed.
13. Release tourniquet and withdraw needle from vein.
14. Strip blood from donor tubing into bag, mix and allow tubing to refill. Prepare segments by sealing appropriate locations of donor tube.
Precaution: Start seals at needle end of tube and work toward bag.
15. Except Single BLOOD BAG unit, the empty satellite bags may now be used as transfer bags for further processing according to standard component processing techniques.

INSTRUCTIONS FOR BLOOD SAMPLING PORT (BSP)

1. After collecting desired volume to the primary bag, clamp the donor tube above the "Y" connection to the primary bag and cut off the blood flow.
2. Remove the rubber cap from BLOOD SAMPLING PORT and attach blood collection device securely. Alternatively, blood sampling needle can be inserted directly to the attached rubber cap.
3. Bend and break snap-tip and allow the blood to flow to BLOOD SAMPLING PORT.
4. Insert a vacuum test-tube to the attached holder. Undiluted sample blood is collected. Multiple samples can be taken by exchanging test-tubes.
5. After completion of sample blood collection, release tourniquet and withdraw needle from vein.
6. Dispose the part from the donor needle to the "Y" connection of BLOOD SAMPLING PORT by acceptable method for potentially contaminated blood products.



INSTRUCTIONS FOR BLOOD SAMPLING BAG (BSB)

1. Prior to blood collection, open clamp A and close clamp B as indicated in fig. 1.
2. Break the click tip to allow blood from donor to flow towards BSB.
3. After collecting about 30ml of blood, close clamp A and open clamp B as indicated in fig. 2.
4. Seal off the tubing between clamp A and click tip. Proceed to conduct blood sampling using Integrated Blood Sampling Port.

INSTRUCTIONS FOR INTEGRATED BLOOD SAMPLING PORT (IBSP)

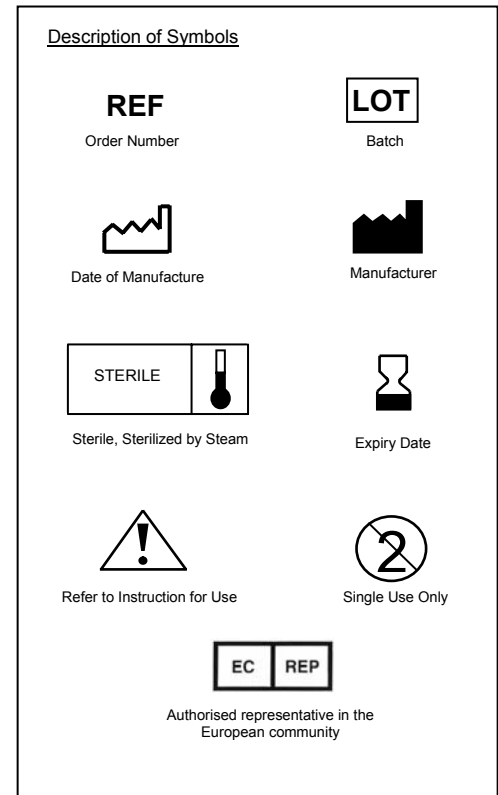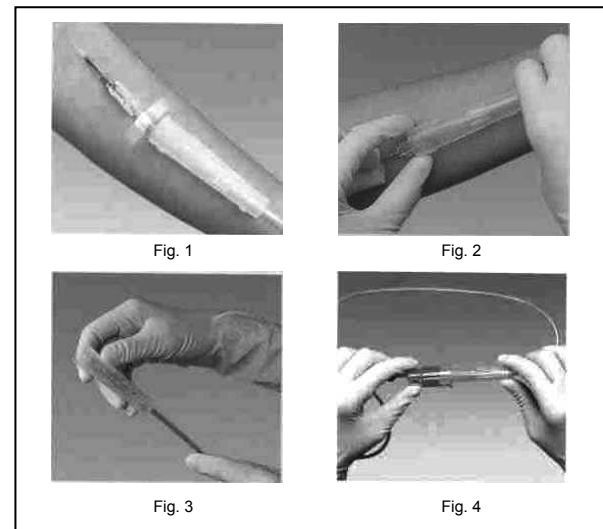1. After collecting desired volume to the primary bag/BSB, clamp the donor tube above the "Y" connection to the primary bag/BSB and cut off the blood flow.
2. Bend and break click tip (if any) and allow the blood to flow to INTEGRATED BLOOD SAMPLING PORT (Tube holder).
3. Insert a vacuum test-tube to the attached holder. Undiluted sample blood is collected. Multiple samples can be taken by exchanging vacuum test-tubes.
4. After completion of sample blood collection, release tourniquet and withdraw needle from vein. (Fig. 1)
5. Insert the needle into the needle protector by sliding it along the donor tube until needle is firmly locked. (Fig 2 & Fig. 3)
6. Insert the needle protector into the INTEGRATED BLOOD SAMPLING PORT (Tube holder) and discard. (Fig. 4)
7. Dispose the part from the donor needle to the "Y" connection of INTEGRATED BLOOD SAMPLING PORT by acceptable method for potentially contaminated blood products.

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